

## Metoject<sup>®</sup> PEN frequently asked questions

### Why won't my injection start with the Metoject<sup>®</sup> PEN?

Metoject<sup>®</sup> PEN contains a safety device which requires sufficient pressure to be applied to the skin to unlock the yellow release button. The injection site surface must be firm throughout the process. Once activated, maintain pressure during the injection without moving the PEN for at least 5 seconds.

### What should I do if there is a drop of liquid on the tip of the needle?

Occasionally a small drop of liquid may be visible on the tip of the needle once you have removed the protection cap. This is completely normal and nothing to worry about.

### How many doses are in the Metoject<sup>®</sup> PEN?

The PEN is a single use device and will deliver only one dose. The PEN device should be disposed of after administration.

### How do I store Metoject<sup>®</sup> PEN?

Metoject<sup>®</sup> PEN is stored at room temperature (less than 25°C). It can be kept in the fridge (minimum temperature 2°C) but should be allowed to warm to room temperature before administration. It can be stored for up to two years from date of manufacture.

For further information and administration instructions, please refer to the full adult or parent and child administration guidelines or speak to your pharmacist, nurse specialist or GP.

